

# ComCierge CareTeam Software (iPad or iPhone) Setup

## 1. Download the ComCierge CareTeam App

Search for “HatchMed” on the App Store, and download the “ComCierge CareTeam” app (blue icon with a cross in the middle).


*ComCierge ComCierge is only available on iPads and iPhones.*



## 2. Log in to the ComCierge CareTeam App

Use the provided log in on the home screen, it will follow the format of “charge@facility.com”.

*An internet connection is required to log in and access the app's features.*

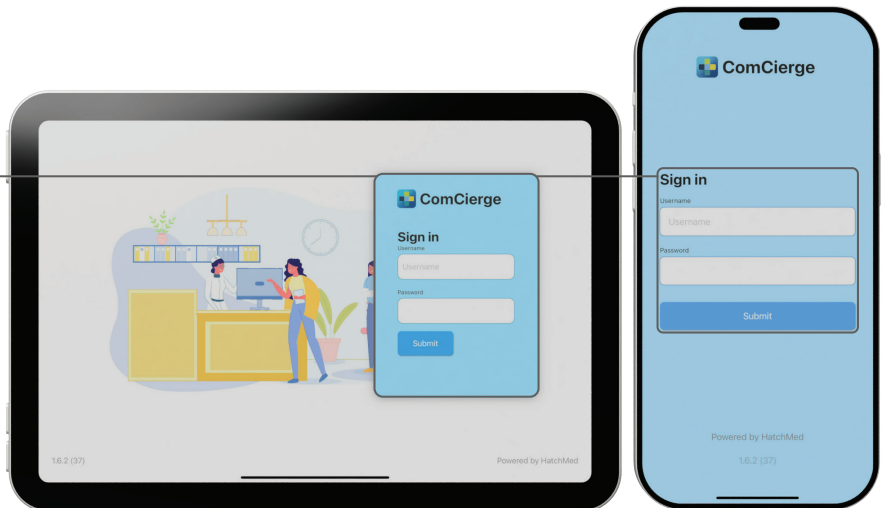

## 3. Assign rooms to supervise for this nurse

Click the “Bays” tab on the right, click “Manage Bays”, and select the appropriate patient rooms.

*Any requests placed on non-assigned ComCierge Patient rooms will not display.*

