

HighLight

by HATCH+MED

Instant Visibility

At-a-glance confirmation of bed connection status, ensuring quick, informed decisions—no second-guessing safety.

Automated Workflow

Seven input/output ports connect any patient room equipment to the nurse call system for accessibility.

Proactive Safety

Real-time alerts ensure beds stay connected, reducing response times and enhancing patient safety.



HighLight gives care teams instant visibility into bed connections while seamlessly linking hospital equipment to the nurse call system—no extra steps, just clear, reliable updates.



info@hatchmed.com

HATCH+MED



844-428-2463



Bed Status LED

Programmable Colors

Dynamic Button

Custom Functions

Light Sensor

Auto-dims LEDs

Primary 37 Pin


Bed Connection

Secondary 37 Pin

Extra Bed Connection

Output 37 Pin

Nurse Call Connection

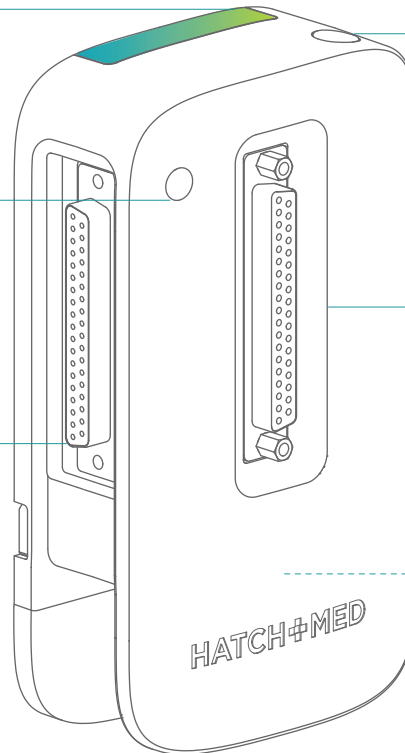


Priority Call 1/4" Expansion Jack

Routine Call 1/4" Expansion Jack



Power Connection

Use only provided 12V 3A power supply

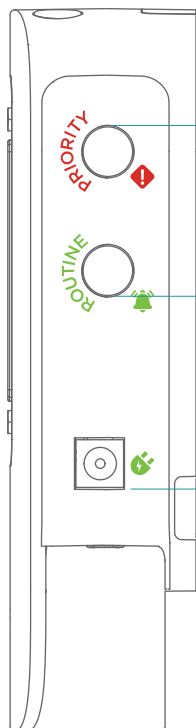

Equipment Required


Tools for mounting HighLight to a single gang opening or directly to the headwall. Drywall & adhesive mounting hardware is provided.

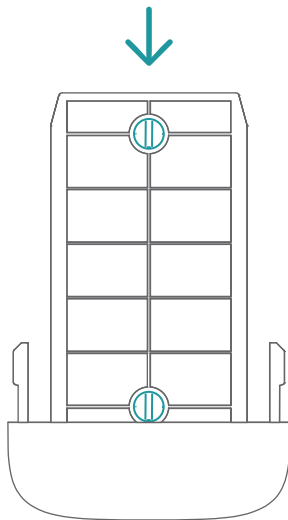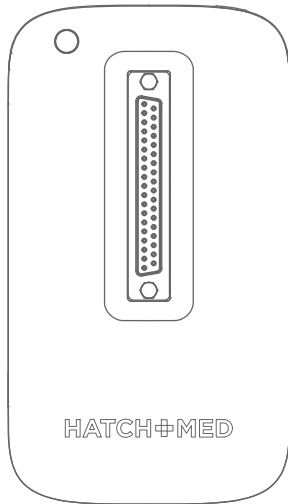

Mounting

Secure the mounting bracket and clip the HighLight into place.

Position appropriately to be visible by the care team & avoid bed impacts.

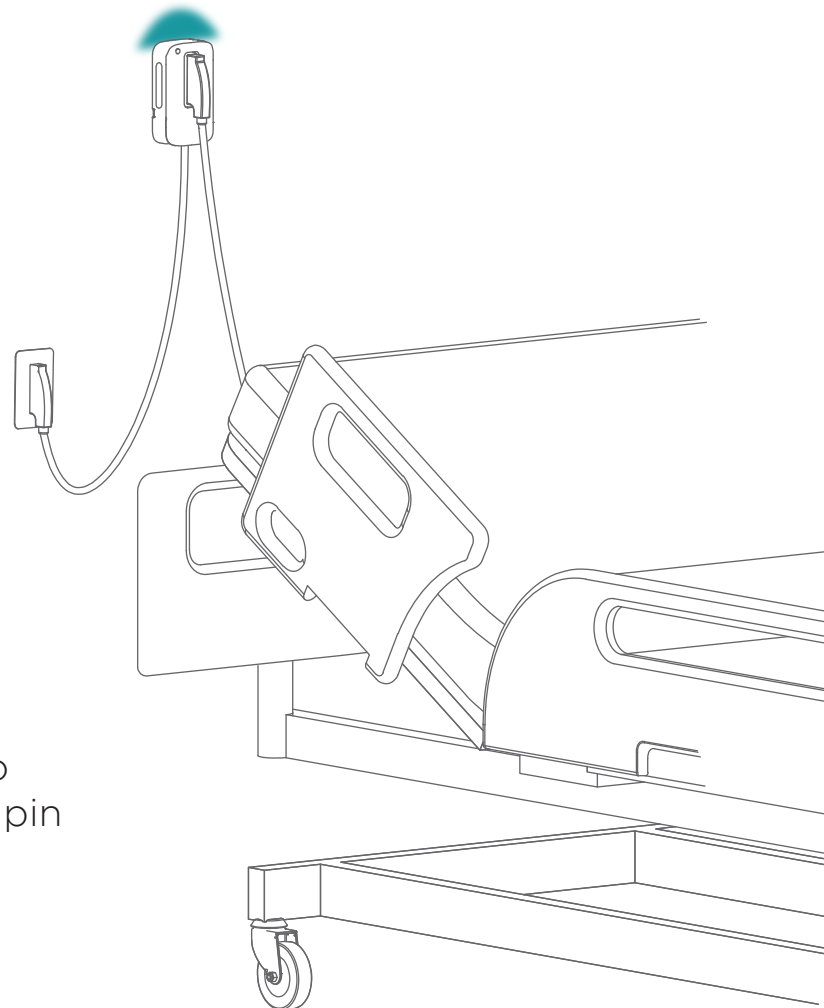

Connecting

Connect the bottom 37 pin port to the nurse call system, the front 37 pin to the bed, as well as any auxiliary connections as needed.



HighLight *HMNE-3277*

HighLight features seven input/output ports for seamless bed and equipment connectivity, with illuminated indicators for instant status updates.

Size & Weight

Weight	0.3 lb
Dimensions	3.3" W x 5.0" H x 1.3" D

General Info

Power Draw	12 VDC @ 100 mA	Inputs	Programmable workflow button
LED Illumination	RGBW LED panel, auto brightness & support for varied patterns & status. Green / Red default connected / disconnected	Cleaning	Stable with standard hospital surface cleaners
Material	ABS UL V0 rated, UV stabilizing & antimicrobial additives	Interfaces	Front: Primary 37 pin bed connection Left: Secondary 37 pin bed connection Right: Priority & Routine 1/4" Accessory ports Bottom: 37 pin connection for nurse call output
Data Transmission	SmartHub connection port allows for all data & nurse call events to be monitored & tracked via API.	Regulatory	FDA Registered
Call Types	Emergency call & bed exit		

